

CorDx Tyfast RSV Rapid Test



Backed by the most recent U.S. clinical evidence, ready for today's post-COVID reality

Features and Benefits

- **Rapid Results** – Provides accurate results at the **point-of-care** in just 15 minutes.
- **Noninvasive** – Uses a simple **anterior nasal swab** to improve patient comfort and compliance.
- **Quality You Can Trust** – U.S.-manufactured with integrated quality controls and dependable customer support.
- **Reliable for At-Risk Groups** – Rapid RSV detection for infants, children (6 months-6 years old), and adults over 60 years old.



What's Inside


- 25 Test Devices
- 25 Reagent Tubes & Dropper Tips
- 25 Sample Collection Swabs
- 1 Positive Control Swab
- 1 Negative Control Swab
- 1 Instructions for Use
- 1 Quick Reference Instructions

Performance Specifications:


	Sensitivity*	Specificity*	Sample Type	Storage Temperature	Time to Result	Shelf Life
RSV	83.8%	99.6%	Anterior nasal swab	2-30°C/ 36-86°F	15 minutes	18 months (Extension Expected)

*Combined prospective clinical RT-PCR comparator study.


Seamless Sample Processing with Minimal Hands-On Time

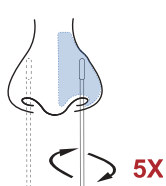


Step 1:
Collect the anterior nasal swab sample.

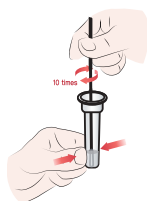

Step 2:
Stir the swab in Reagent tube at least **10 times**.

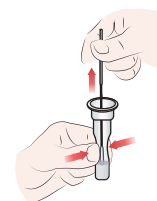

Step 3:
Remove the swab while squeezing the sides of the tube.

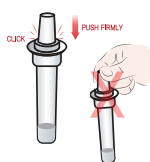

Step 4:
Attach provided dropper tip to close the reagent tube.



Step 5:
Dispense 2 drops of sample into the sample well of the test device.



Step 6:
Read the results at 15 minutes visually.

Ordering information:

Description	CorDx Product #	Package Configuration	Dimensions (L*W*H) & Weight	CPT Code
CorDx Tyfast RSV Rapid Test	ACT21056-25	25 Tests/Box	Kit box: 8.4" x 4.9" x 3.7" (0.8 lbs) Case: 22" x 22" x 16" (37.6 lbs) Pallet: 45" x 45" x 58" (478 lbs)	87807QW

* Test kit does not provide a timer, tube rack for specimens and any necessary personal protective equipment.



- ☎ +1 (800) 711-5355
- ✉ info@cordx.com
- 🌐 CorDx.com
- 📍 5900 Windward Pkwy, Alpharetta, GA 30005